

Alpron 1% oplossing voor continu gebruik in dentale unit

Mixed Solution(ml)	Alpron	Water	Mixed Solution (Ltr)
500 ml	5 ml	495 ml	0.5 Ltr
750 ml	7.5 ml	742.5 ml	0.75 Ltr
1000 ml	10 ml	990 ml	1.0 Ltr
1500 ml	15 ml	1485 ml	1.5 Ltr
2000 ml	20 ml	1980 ml	2.0 Ltr
2500 ml	25 ml	2475 ml	2.5 Ltr
3000 ml	30 ml	2970 ml	3.0 Ltr
3500 ml	35 ml	3465 ml	3.5 Lt
4000 ml	40 ml	3960 ml	4.0 Ltr
4500 ml	45 ml	4455 ml	4.5 Ltr
5000 ml	50 ml	4950 ml	5.0 Ltr

Continu in systeem laten, dus bij gebruik van Alpron moet fles op de unit blijven zitten en niet gedroogd worden einde werkdag.

Alle handstukken gedurende 1 minuut doorspoelen in de ochtend en aan einde werkdag

Tussen patiënten gedurende 30 seconden alle handstukken doorspoelen.

Wekelijks combineren met Bilpron reiniging:

1. Fles legen, spoelen met schoon water en dan vullen met 250 ml Bilpron
2. Bilpron door alle leidingen spoelen
3. Over het weekend in situ laten (of tenminste gedurende 12 uur [ALPRO | Bilpron \(alpro-medical.de\)](https://alpro-medical.de/))
4. Spoel de fles volledig leeg door de leidingen en verwijder fles van de unit
5. Spoel de fles met schoon water
6. Vul de fles met 1% Alpron oplossing
7. Spoel alle leidingen door met Alpron oplossing, zodat alle Bilpron uit de leidingen is, gebruik hiervoor tenminste een halve unit fles Alpron oplossing
8. Vul de fles daarna weer aan met Alpron 1% oplossing

Bovenstaand schema kan in aangepaste vorm gebruikt worden als de unit gedurende langere tijd (>72 uur) ongebruikt blijft. Dan stap 3. Verlengen tot maximaal 3 maanden.

[How to use Bilpron and FAQS - Aura Infection Control \(aiconline.co.uk\)](https://aiconline.co.uk/)

[How to use Alpron - Aura Infection Control \(aiconline.co.uk\)](https://aiconline.co.uk/)

[Alpron Efficacy - A 6-year study - Aura Infection Control \(aiconline.co.uk\)](https://aiconline.co.uk/)

[ALPRO | Bilpron \(alpro-medical.de\)](https://alpro-medical.de/)

[ALPRO | Alpron \(alpro-medical.de\)](https://alpro-medical.de/)