DERMICOAT

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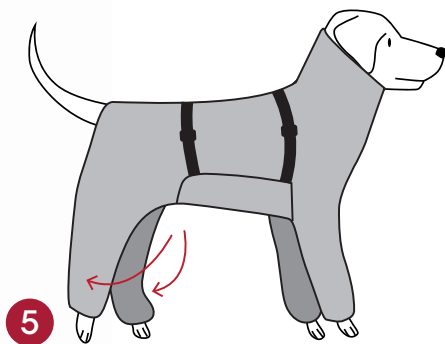
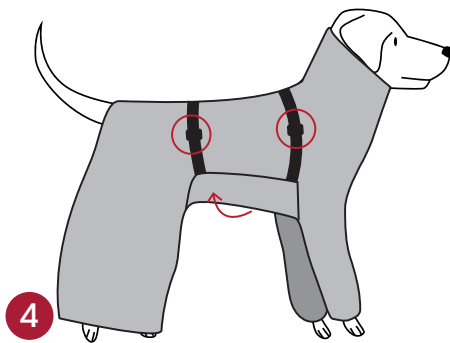
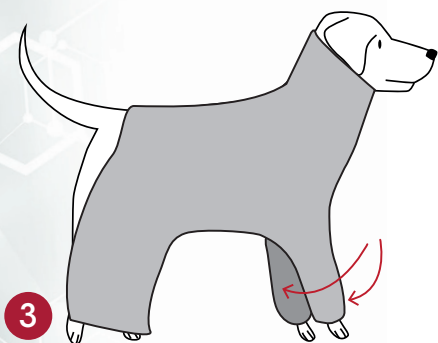
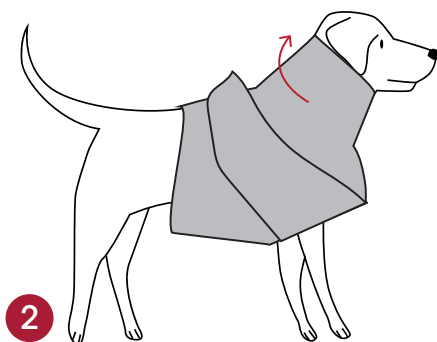
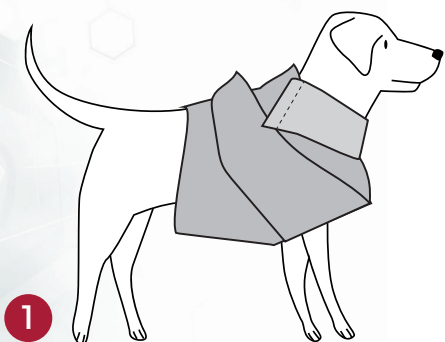
EN | User manual

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Maxani DermCoat

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Maxani DermCoat

For effective and comfortable coat treatment

The Maxani DermCoat is specially developed to enhance the effect of medicinal shampoos. Thanks to the unique inner layer, the shampoo stays well on the skin while your dog can move freely. This makes treatment easier for you and more comfortable for your pet.

Instructions for use

- 1 Pull the DermCoat over your dog's head.
- 2 Fasten around the neck.
- 3 Insert the front legs through the openings.
- 4 Attach the belly flap on the back.
- 5 Finally, close the hind leg flaps.

Washing instructions for the DermCoat

Wash at 30°C.
Not suitable for tumble drying.

Note

Do not let your dog wear the DermCoat in direct sunlight or warm weather due to the risk of overheating.

Contact

www.maxani.eu | info@maxani.eu | +31 (0)30 751 70 67

Why use Maxani skin shampoo with the DermCoat?

The DermCoat is designed to optimize the effectiveness of skin shampoos. Maxani offers a range of shampoos tailored to different skin types and needs. Your veterinarian will be happy to advise you on the best choice.

Which shampoo are you using?

- 🚩 **Pyomax** - For disturbed skin flora or odor problems
- 🚩 **AtopOat** - For itchy or allergic skin
- 🚩 **Honey Complex** - For dry to very dry skin
- 🚩 **SeboSilk** - For oily, flaky skin
- 🚩 **Rescue** - To support redness or discomfort
- 🚩 **Regular** - For regular care of normal or sensitive skin

Washing instructions

Unless otherwise advised by your veterinarian:

1. Thoroughly wet the coat (lukewarm water).
2. Dilute the shampoo (1:5), or apply undiluted to problem areas.
3. Massage in well – also between toes, around the tail, and in skin folds.
4. Rinse and repeat steps 1, 2, and 3.
5. Put on the DermCoat.
6. Leave on for 10 minutes.
7. Remove the DermCoat.
8. Rinse the coat thoroughly until the water runs clear.
9. Pat dry with a soft towel – do not use a hairdryer.

Tip: Do not leave a layer of water in the tub – this dilutes the shampoo and reduces effectiveness.

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