Fassisi® Bovine IgG

FASSISI® The rapid measurement of bovine IgG in blood samples

The Fassisi® Bovine IgG test has been developed for the determination of the concentration of bovine IgG in blood samples of calves. The IgG concentration in blood samples is an indicator of the health status of calves and gives a first indication of the future productivity and the future value of the animal. In general, one can assume that a successful passive transfer of bovine IgG to the newborn calf is realized, if a concentration of IgG in serum is > 10 mg/ml.

With a concentration in the range of 5–10 mg/ml a PFPT (partial failure of passive transfer) can be considered, while concentrations of < 5 mg/ml indicate a FPT (failure of passive transfer). In this case, immediate measures should be taken to provide the animal with adequate immunological protection and to protect it against possible infections.

Please note before use:

- Use a new test cassette for every individual test.
- Test cassettes are for single use only
- For veterinary use only.
- Use only the original test components provided in the Fassisi kit.
- Use the test cassette within 60 minutes after opening the pouch.
- The cassette must be in a horizontal position on a flat surface while the test is performed.
- Note the amount of sample material needed. An incorrect number of drops or undersized drops may lead to false test results.
- Consider the results as invalid after the specified read out time.
- Do not use the test after the expiration date printed on the pouch.
- Dispose of all contaminated materials properly. Disinfect the work area after the test execution.

Storage of the test kit:

The test kit must be stored between 2–30°C.

Choice of sample material

Serum and plasma

Recommended sample material is a freshly collected serum or plasma to achieve the highest detection sensitivity.

Separate the serum or plasma from whole blood as quickly as possible, the specimen should be clear and non-hemolyzed.

Whole blood

A whole blood sample should be used as quickly as possible. Heparin blood and EDTA blood may also be used. Hemolyzed samples should not be used for testing.

Note: The use of whole blood samples have a lower detection sensitivity. In case of an unclear test result with whole blood, the test should be repeated with a serum or plasma sample.

Literature:

Suh et al. (2003) "Differences in the serum immunoglobulin concentrations between dairy and beef calves from birth to day 14 of age". J. Vet. Sci. 4(3): 257–260.

National Animal Health Monitoring System (1992) "Dairy herd management practices focusing on preweaned heifers". USDA, Animal and Plant Health Inspection Service, Veterinary Services, Fort Collins, Co.

Noordhuizen, J. (2012) „Dairy Herd Health and Management. A guide for veterinarians and dairy professionals“, Context Products Ltd, Leicestershire, England.


Sensitivity and Specificity

Comparison method: Radial immunodiffusion (RID)

Study 2018
N=120

	Sensitivity	Specificity
Bovine IgG	95,52 %	96,23 %

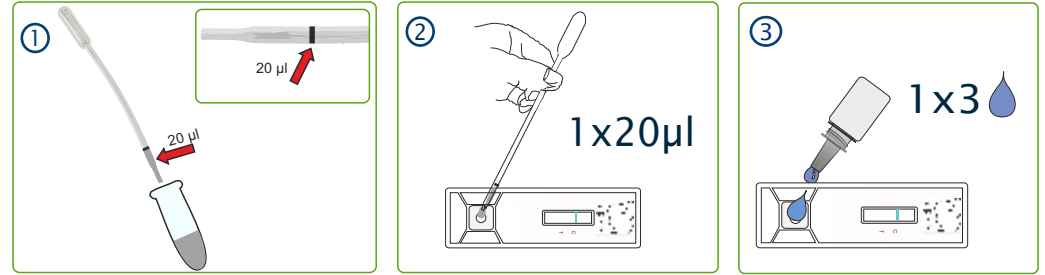Instruction Manual

Test procedure

Preparation:

Open the pouch, remove the test cassette, place the test cassette on a flat surface and unscrew the bottle of reagent and place it aside.

Good to know: In the reaction field you can see a green line in the control line region before starting the test. These are used for quality control and will be washed away by the sample liquid during the test.

Useful tip: If the fluid does not run up the test strips after 60 seconds, add an additional drop of the reagent into the sample well.

① Using the marked 20 µl pipette, aspirate the sample material up to the 20 µl mark.

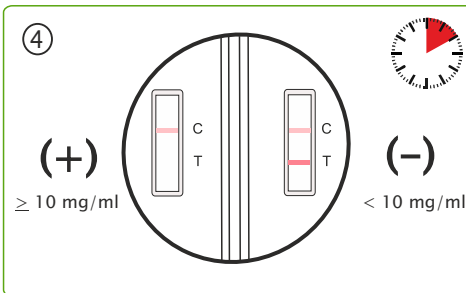
② Now add this material to the sample well and allow the 20 µl to be drawn in.

Avoid the formation of air bubbles.

③ Take the opened bottle of reagent and add 3 drops of reagent to the sample well. After a few seconds, the liquid starts to run over the test strip.

Test result

The result of the test can be read after 10 minutes.

(+) No test line is visible: ≥10 mg/ml

The IgG concentration is high and in the normal range.

A successful transfer of passive immunity can be assumed (≥10 mg/ml).

(-) A test line is visible: <10 mg/ml

If a strong test line becomes visible, the transfer of passive immunity (TPI) was poor and up to deficient, the concentration of IgG in the sample is clearly too low.

If a weak test line becomes visible, the transfer of passive immunity was not optimal, the concentration of IgG in the serum is < 10 mg/ml.

Invalid:

If no line appears in the C-line area the test is invalid and should be repeated.

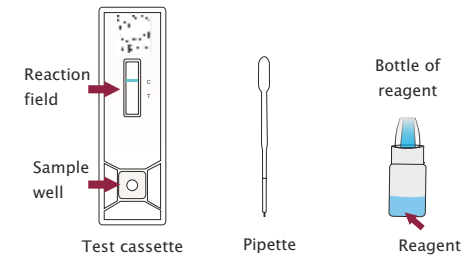
Good to know:

The more the test line is visible, the lower the IgG concentration is to be classified. If the test line is clearly visible, there is a deficient transfer of passive immunity. The IgG concentration is too low. Action is required.

DISCLAIMER: The entire risk of liability in connection with the use of this product rests with the purchaser. The manufacturer assumes no liability for indirect, special or consequential damages of any kind resulting from the use and test evaluation of this product.

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Symbols

- Observe product information
- Contents
- Protect against light
- For professional use only
- Lot number
- Protect from humidity
- For one-time use only
- Shelf life
- Manufacturer
- Storage temperature: +2°C to +30°C

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