USER MANUAL – PetAnesth

Anesthesia Circuit Holding Device – Veterinary Use

1. Description

PetAnesth is a holding device designed to secure anesthesia tubing during surgical procedures. It is suitable for use in both domestic carnivores and exotic pets.

2. Components

- Flexible silicone-coated arm with 360° orientation.
- Table clamp mounting system adaptable to surfaces up to 5 cm thick, to the arm of an anesthesia machine (gas machine), or to the bars of kennel cages (oxygen therapy, inhalation...).
- "Paw" Circuit Holder: 3D-printed terminal part with slots to secure the anesthesia circuit.
- Adjustment knob to set rigidity.

3. Installation

1. Place the clamp at the desired location.
2. Tighten firmly until full stability is achieved.
3. Check that no lateral movement is possible.
4. Adjust the flexible arm before positioning the circuit.

4. Positioning the Anesthesia Circuit

1. Gently insert the anesthesia tubing into one of the slots of the Paw Holder.
2. Adjust the arm position to avoid any traction.
3. Ensure there is no kinking or compression of the circuit.
4. Make sure the circuit does not obstruct access to the patient.

5. Maintenance and Cleaning

- Clean after each use with a mild, non-corrosive detergent.
- Do not immerse the device.
- Do not autoclave.
- Regularly check the condition of joints, the Paw Holder, and the clamp.

6. Precautions for Use

- Never use the device to restrain an animal.
- Always check stability before use.
- Avoid excessive traction on tubing.
- Do not use the device if visible damage or instability is present.

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